

AAC app cheat sheet

TouchChat WordPower 42

Vocab: Copy and select other TouchChat page sets here

Pronouns: Common pronouns are coded yellow. More people words can be found in the people category

Message Window: Press to say message

Menu: Go here to change look of the board, edit cells and pages, and more.

Arrow X: Delete whole message

ABC/123: Access to a keyboard and the FIND WORD feature

Home: The yellow home button on other pages will always bring you back to your home page

Fringe Words: Words organized by category. The single and double arrows mean the cell links to another page

Extra Words: Connecting words, little words, and transition words

Action Words: Common action words coded green

Clear: Erases what is in the message window

Vocab						Menu
PEOPLE 	QUESTIONS ?	PLACES 	SOCIAL 	TIME 	GROUPS 	ABC 123
I	can	drink	eat	come	good	DESCRIBE
my	do	finish	help	feel	a	more
it	have	like	listen	go	the	that
you	is	want	watch	play	stop	EXTRA And It The WORDS Up
your	don't	.		work	clear	

TouchChat® is a trademark of PRC-Salttillo

Forbes AAC
181 Illinois Ave. South
Mansfield, OH 44905
419.589.7688
fax 419.589.5146



forbesaac.com