


AAC app cheat sheet


WinSlate Grid 3

Teletherapy Assessment **Quick Tip**

Grid Explorer: This is a dashboard containing all the grids the user has added. Customize this page with folders and grids. Available grids include symbol based grids (Core Word, Super Core, Symbol Links, Symbol Talker, Word Power 25, 26, and 100, PODD(\$)); text based grids (Fast Talker, and Text Talker); specialized grids (Alpha Core, and Vocabulary for Life); Grids that provide additional support (educational activities, literacy actives, environmental controls, templates and examples, etc.)

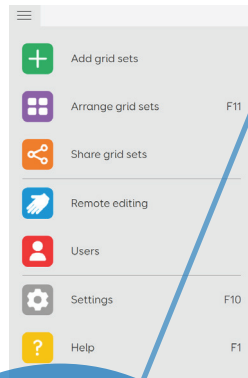

Current
Username: Change
username and
picture in
settings

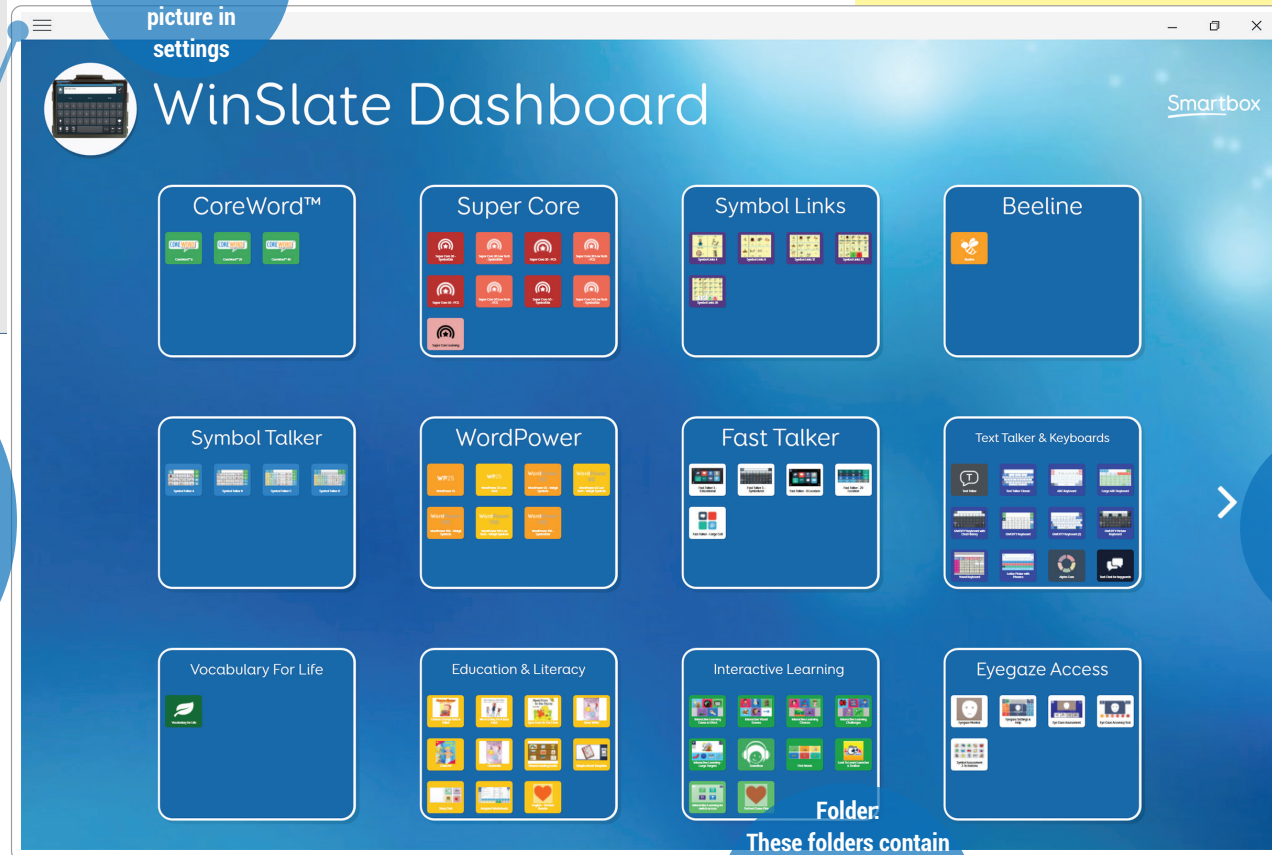

Use three lines
in upper left to access drop
down menu. Here you can add
a grid set, arrange grid sets
(sort or delete), share a grid set,
remote edit, customize user(s),
settings (Access, Speech,
Writing, Accounts, Contacts,
Web, Phone, Computer,
Environmental control,
Licenses), and Help

Right Arrow:
Navigate to
second page
with more grid
choices

These folders contain
grids that the user has
added. Add or remove
grids using menu drop
down and customize to
user preferences

forbesaac.com/teletherapy-assessment-program

WinSlate Dashboard® Forbes AAC
Grid 3® is a trademark of Smartbox

Forbes AAC
181 Illinois Ave. South
Mansfield, OH 44905
419.589.7688
fax 419.589.5146



forbesaac.com