

AAC app cheat sheet

Snap Core First

Core Words & Home: Use to return to the home page of core words

Search Icon: Use to find and navigate to a word

Cloud Sync: Use to sync to account

Settings: Use to edit pages, buttons, preferences, etc.

Speak: Use to speak message

QuickFires: Common words and phrases for fast communication

Topics: Messages Organized by Category: Each topic offers additional supports i.e., visual timer, social story, etc.

Keyboard: ABC keyboard with word/symbol prediction

Dashboard: Operational controls

Sharing: Share contents of message window with connected accounts (text, email, and social media)

Bilingual Spanish: Use to toggle between English & Spanish pagesets

Blue Column: Question words

Green Column: Action words

Orange Column: Common little words

Pink Column: Additional categories of words

Yellow Column: Pronouns

Grey Column: Function categories - punctuation and word endings

forbesaac.com/teletherapy-assessment-program

Snap Core First® is a trademark of Tobii Dynovox

Forbes AAC
 181 Illinois Ave. South
 Mansfield, OH 44905
 419.589.7688
 fax 419.589.5146



forbesaac.com