

AAC app cheat sheet

Clicker Communicator (Core 2)

Folder Icon: Use to set level of vocabulary (iPad -> Vocabulary Set -> Clicker Core 1, 2, or 3)

Star Icon: Where favorite phrases are stored

Message Window: Composed message will appear here as it is typed

Question Mark: Help Menu

Grid Icon: Opens editing menu

Gear Icon: Use to access settings for Message Area, Grid, Speech, Predictor, Accessibility, Symbols, and Sharing

Circle X: Delete message

Arrow X: Delete word

Buttons with a red corner: are folders that link to another page with more words and phrases

Back Arrow: Use to return to the previous screen

Home Icon: Use this icon to return to home page

Indicates current page

Home

Clicker Core 2

Pull down tab for more tools (see above)

Chat

Actions

Describe

Food

Places

Clothes

School

Time

Help

I **he** **want** **do** **more** **People**

you **she** **like** **go** **here**

that **it** **is** **in** **again**

what **where** **stop** **to** **not**

Clicker Communicator (Core 2) by Crick Software

Forbes AAC

181 Illinois Ave. South
Mansfield, OH 44905

419.589.7688

fax 419.589.5146



forbesaac.com