

Displaying the iOS pointer during Teletherapy Assessments

You can connect a Bluetooth mouse to your ProSlate or iPad and then display the mouse pointer!

This is tremendously helpful because one of the teletherapy challenges that SLPs now face is not being able to visually display where they are pressing on their iPad screen. While they can easily share the iPad screen, showing the user and family where to press on the screen, has proven to be difficult.

By connecting a Bluetooth mouse and turning on *assistive touch* – you can visually show how and where you are interacting with your favorite app or communication board. **Check out the helpful video that shows how to turn on the settings and see the mouse pointer.**

To use a Bluetooth mouse for feedback, follow these easy steps:

1. Go to **Settings > Accessibility > Touch.**
2. Choose **AssistiveTouch** and toggle it on.
3. Scroll down to **"Pointer Devices"** and tap **"Devices."**
4. Tap on **"Bluetooth Devices"** to start the pairing process.
5. Now set your Bluetooth mouse to discoverable (or pairing mode) and tap on its name when it shows up on your device.
6. From here you can make changes to pointer style, tracking speed, etc.

New! Teletherapy Assessment Program

In response to the COVID-19 Pandemic, Forbes AAC recently launched a Teletherapy Assessment Program to facilitate teletherapy AAC assessments. This innovative program combines loaner AAC devices and a Video Conferencing System to connect SLPs with their end users, **free of charge!** If you are an SLP who needs to do a remote AAC evaluation assessment – we can send an AAC device and video conferencing system directly to your end user. Additionally, we'll help connect your teletherapy session using our HIPPA Compliant video sessions. [Click here to learn more!](#)



[Check out the How-to Video!](#)



Teletherapy Assessment Program

Forbes AAC

181 Illinois Ave. South
Mansfield, OH 44905

phone 419.589.7688

fax 419.589.5146



forbesaac.com
forbesaac.com/virtual-assessment-program