

# AAC app cheat sheet


## Proloquo2Go 5x9 Grid



Press and hold buttons with a colored corner to get a small popup of related words

Folders: Link to a page of words in a category

Tools Popup: Use to access volume controls, keyboard, change language, search for a word, and more

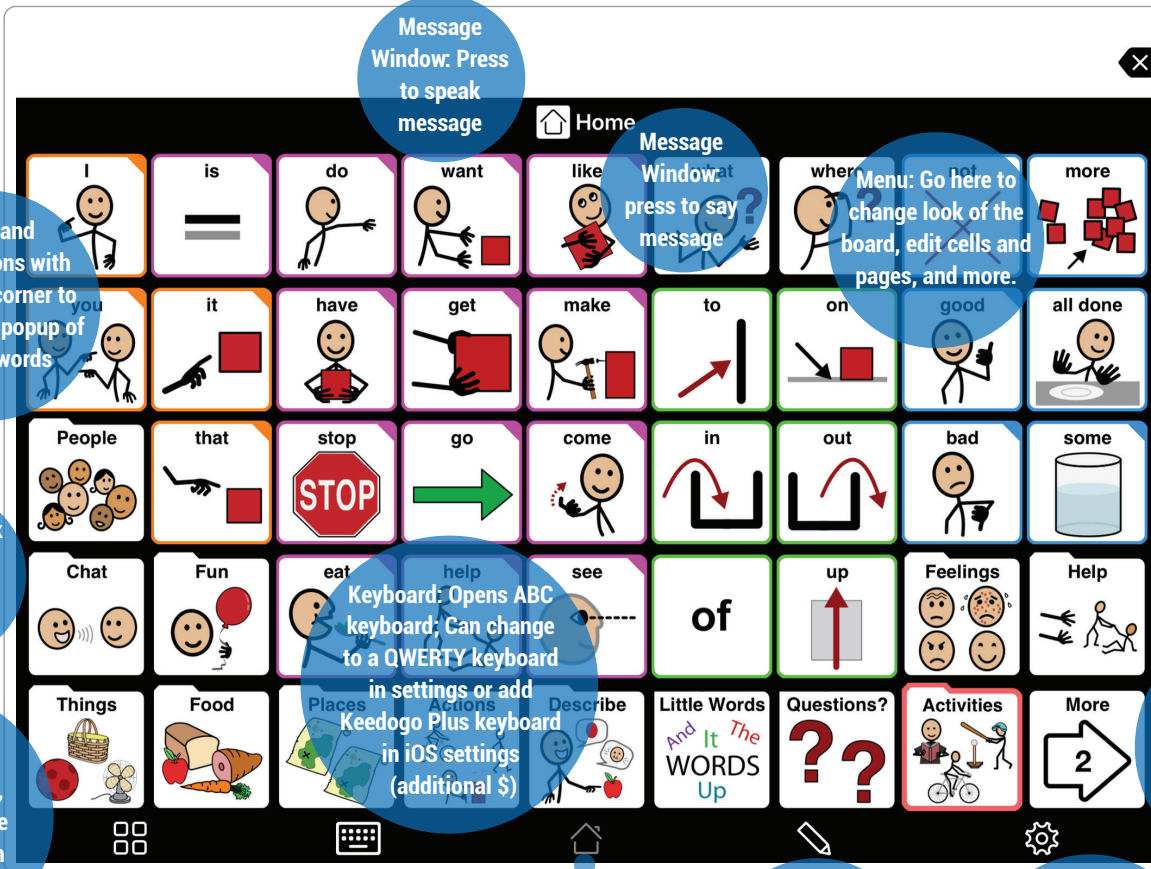

Message Window: Press to speak message

Message Window: at press to say message

Menu: Go here to change look of the board, edit cells and pages, and more.

Keyboard: Opens ABC keyboard; Can change to a QWERTY keyboard in settings or add Keedogo Plus keyboard in iOS settings (additional \$)

Home Button: Press on any page to return to the home page

Edit Icon: Press pencil to get to edit menu to edit buttons and pages

Gear Icon: Opens various features that you can change such as voice, grid size, and more

Arrow X: Delete message one word at a time

More: Place to house additional categories of words such as time, clues, school, religion, home, etc.

Proloquo2go application from Talk to Me Technologies

**Forbes AAC**  
 181 Illinois Ave. South  
 Mansfield, OH 44905  
 419.589.7688  
 fax 419.589.5146



forbesaac.com