

Connecting a device to a WiFi network:



Connecting the AAC device to WiFi allows both the SLP and ATS to remotely access the speech device during your AAC teletherapy assessment. The AAC device screen activity will be broadcast directly during the teletherapy session and when using a WinSlate, the editing and optimizing settings can be done remotely by the ATS or SLP.

Connect a ProSlate device to a WiFi network:

1. Tap the **Settings** icon.
2. Tap **WiFi**.
3. Confirm that WiFi is set to **ON**.
4. Choose your network from the list provided.
5. Enter your network's encryption key if you're prompted and tap **Join**.
6. Tap **On** to automatically join networks your ProSlate has joined previously. If it detects a network it doesn't recognize, it will ask you before it automatically joins the network.
7. Tap the **Home** button to return to the ProSlate home page.

Connect a WinSlate to a WiFi network:

1. Select the Network icon on the taskbar.
2. Select the WiFi network you want.
3. Select Connect, then type the password and follow the instructions.

That's it you're WinSlate is now connected! This icon will appear in the taskbar.

This article was written as a resource to help AAC professionals utilize the new Virtual Assessment Program. To learn more, go to www.forbesaac.com/teletherapy-assessment-program



Forbes AAC

181 Illinois Ave. South
Mansfield, OH 44905

phone 419.589.7688

fax 419.589.5146

