


AAC app cheat sheet

Proloquo4Text

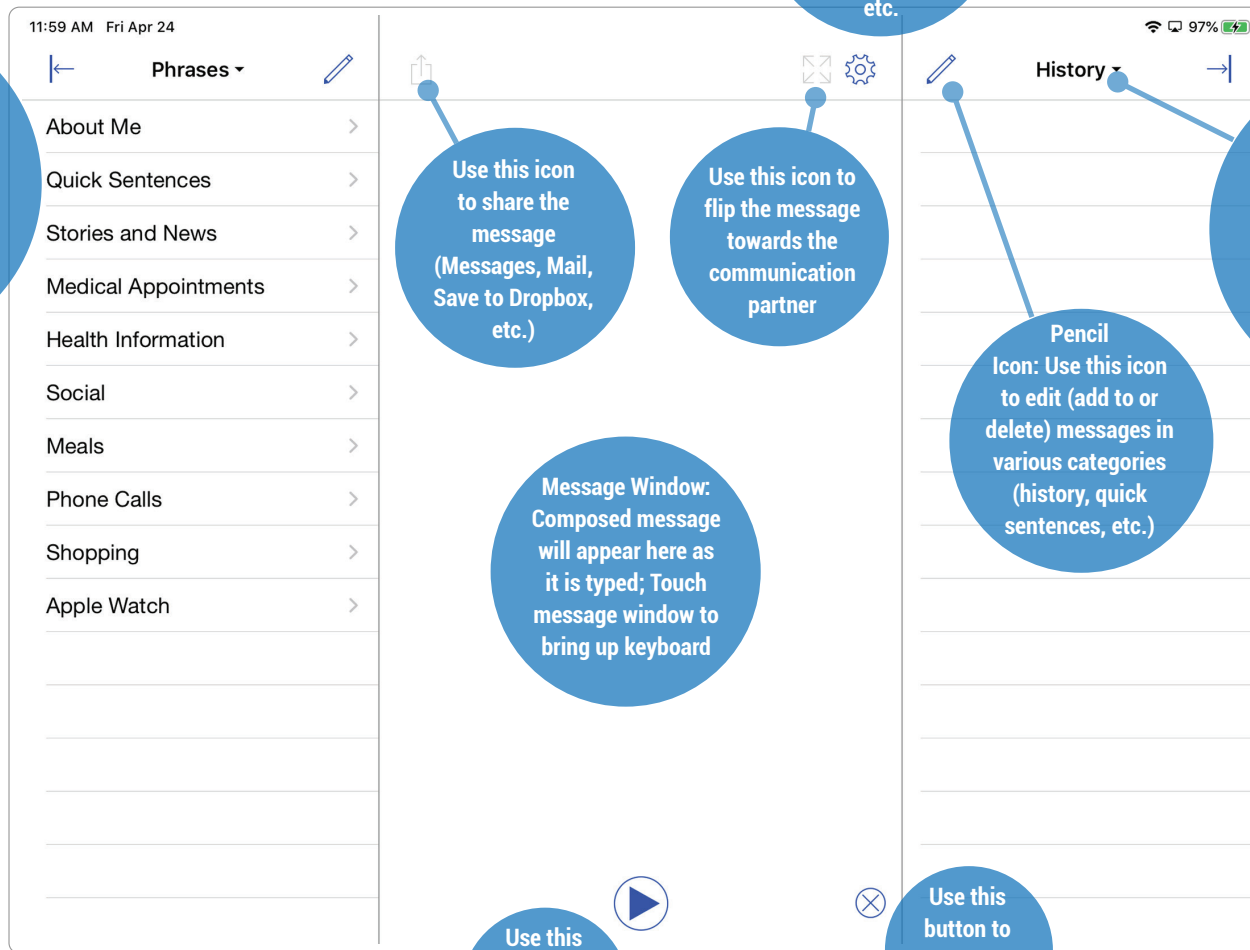

Sentence Prediction: A list of frequently used custom sentences; Use left arrow to collapse column and down arrow to toggle between Phrases, Conversations, and Sentence Prediction

Share Icon: Use this icon to share the message (Messages, Mail, Save to Dropbox, etc.)

Flip Icon: Use this icon to flip the message towards the communication partner

Gear Icon: Opens settings options (appearance, language and speech, prediction, etc.)

Message Window: Composed message will appear here as it is typed; Touch message window to bring up keyboard

Pencil Icon: Use this icon to edit (add to or delete) messages in various categories (history, quick sentences, etc.)

History: A list of recently used phrases appear here; Use right arrow to collapse column and down arrow to toggle between History, Word Prediction, and Quick Talk

Play Button: Use this button to play the message

Eraser Button: Use this button to erase the message

Proloquo4Text by AssistiveWare

Forbes AAC
 181 Illinois Ave. South
 Mansfield, OH 44905
 419.589.7688
 fax 419.589.5146



forbesaac.com