



How to Calibrate the Eye Tracking

1. Go to Menu

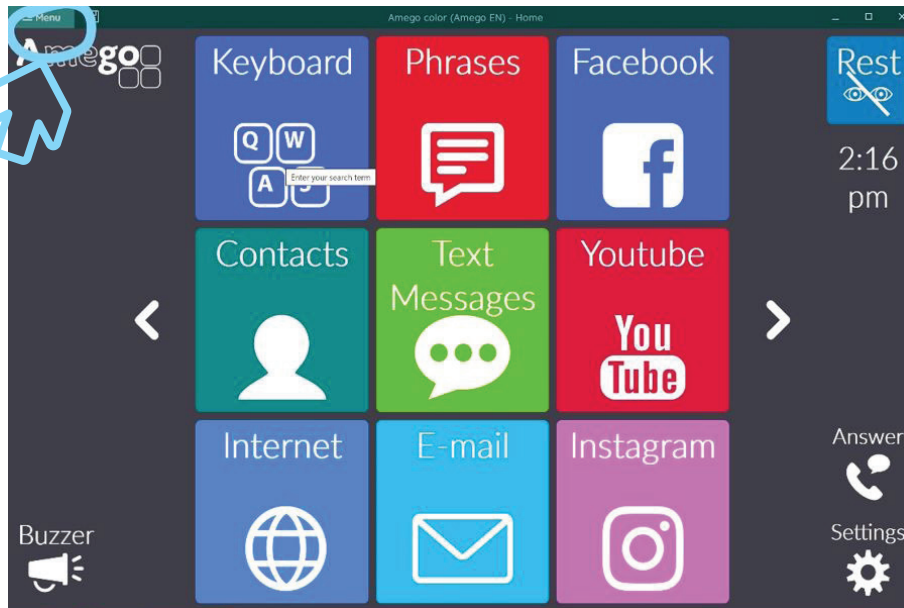

2. Settings

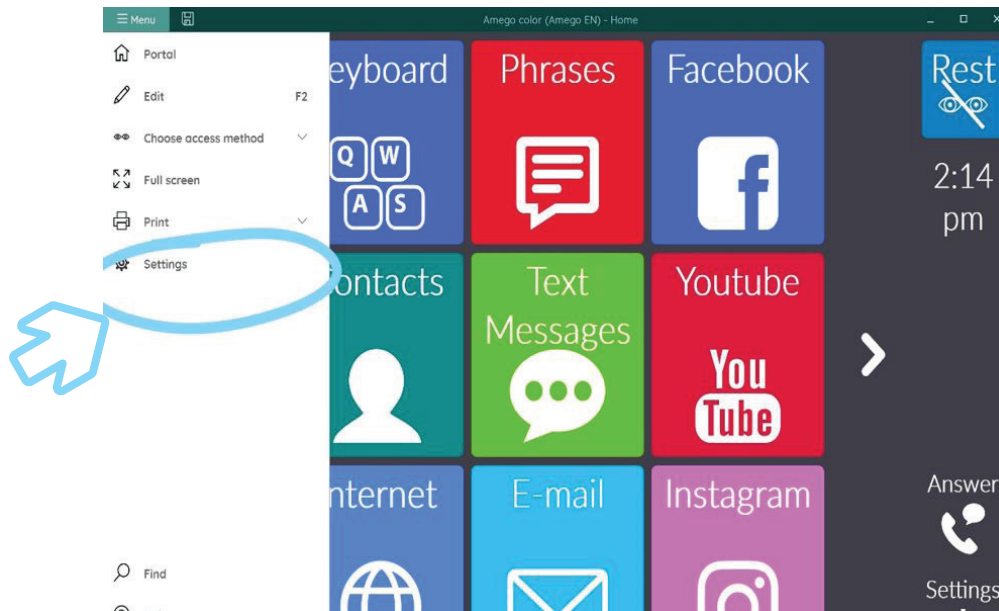


Forbes AAC
181 Illinois Ave. South
Mansfield, OH 44905
phone 419.589.7688
fax 419.589.5146




forbesaac.com

3. Access

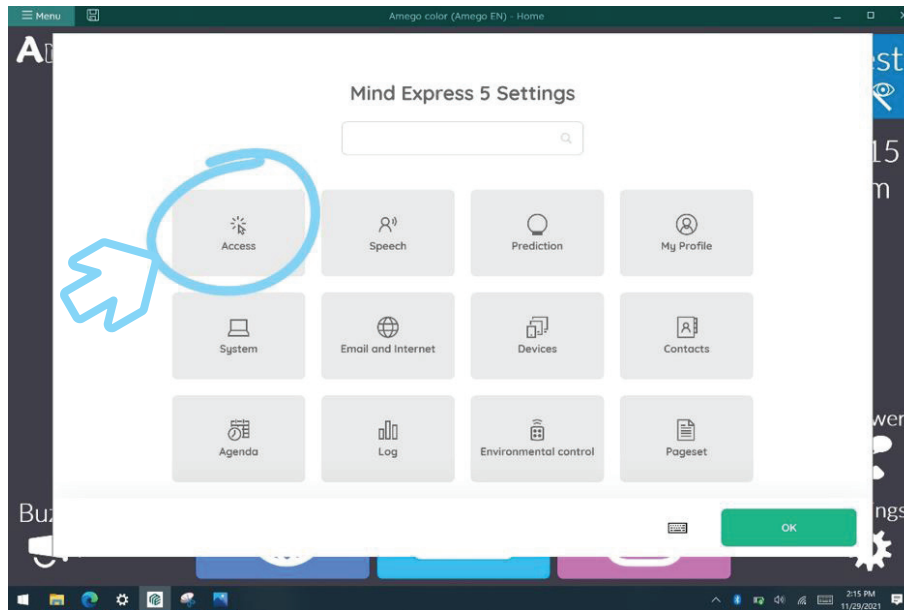

4. Eye Tracking

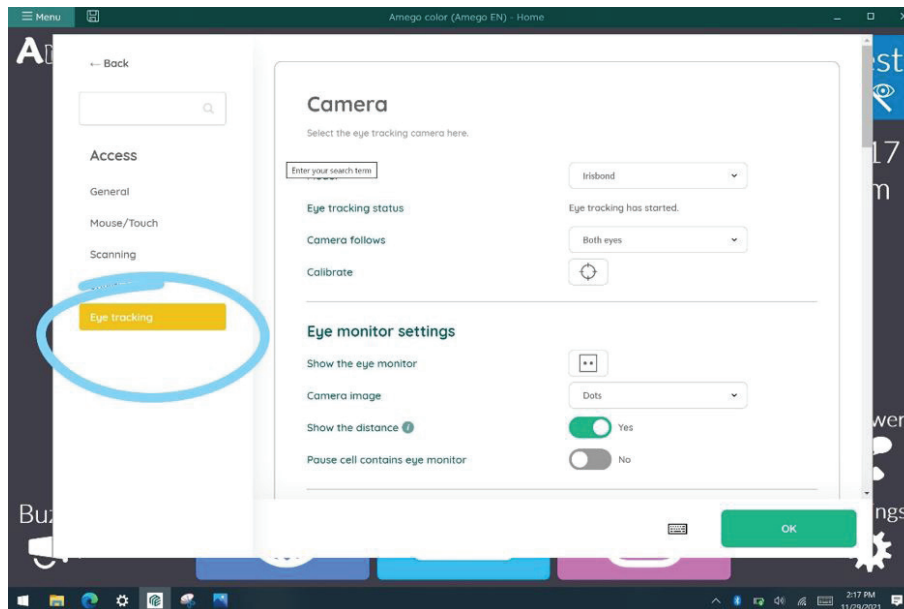


Forbes AAC
181 Illinois Ave. South
Mansfield, OH 44905
phone 419.589.7688
fax 419.589.5146




forbesaac.com

5. Show eye gaze monitor (This can be used to make sure you are in a good position for the camera to see you).

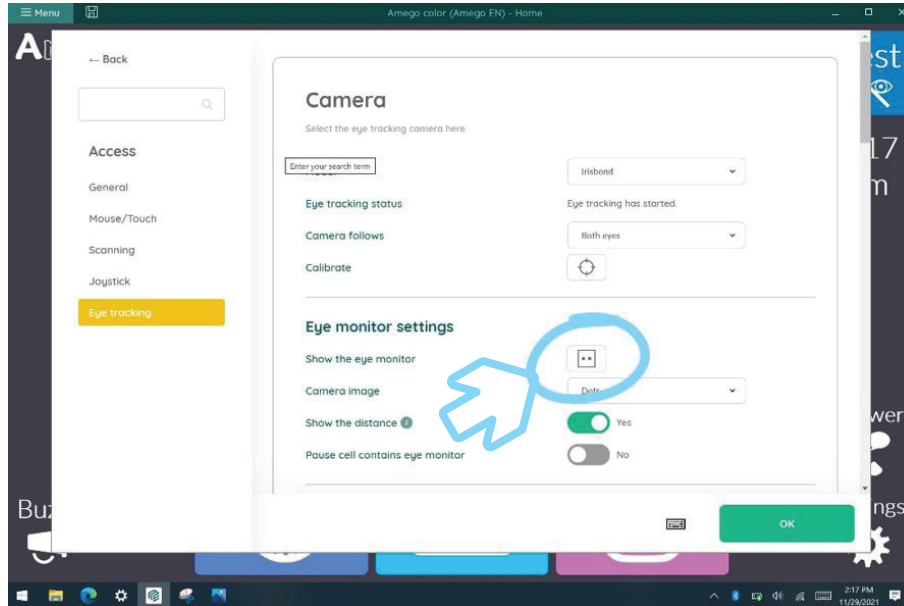

6. Position yourself in the center. Make sure your eyes are centered and filled in black.

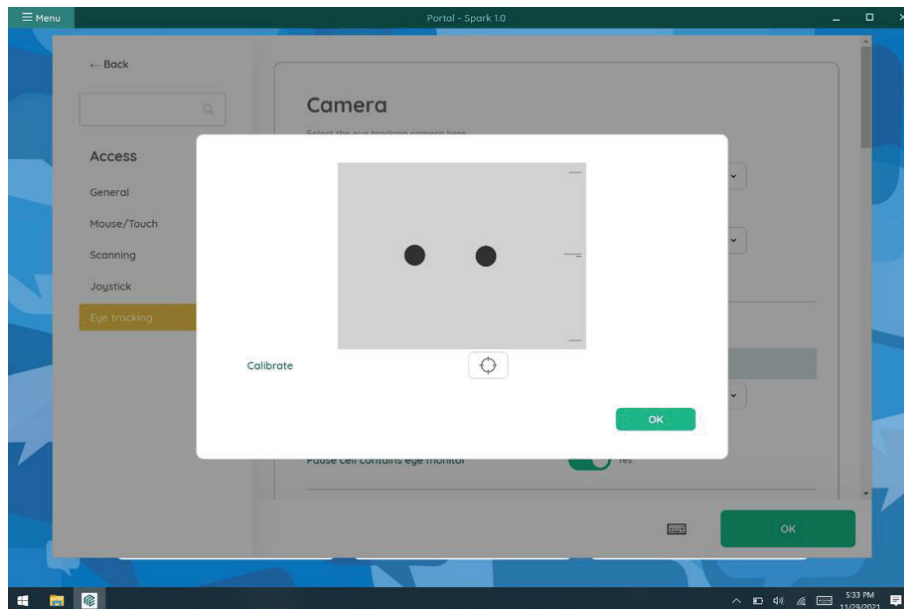


Forbes AAC
181 Illinois Ave. South
Mansfield, OH 44905
phone 419.589.7688
fax 419.589.5146




forbesaac.com

7. Make sure the longer black bar is in or as close to the center on the scale.

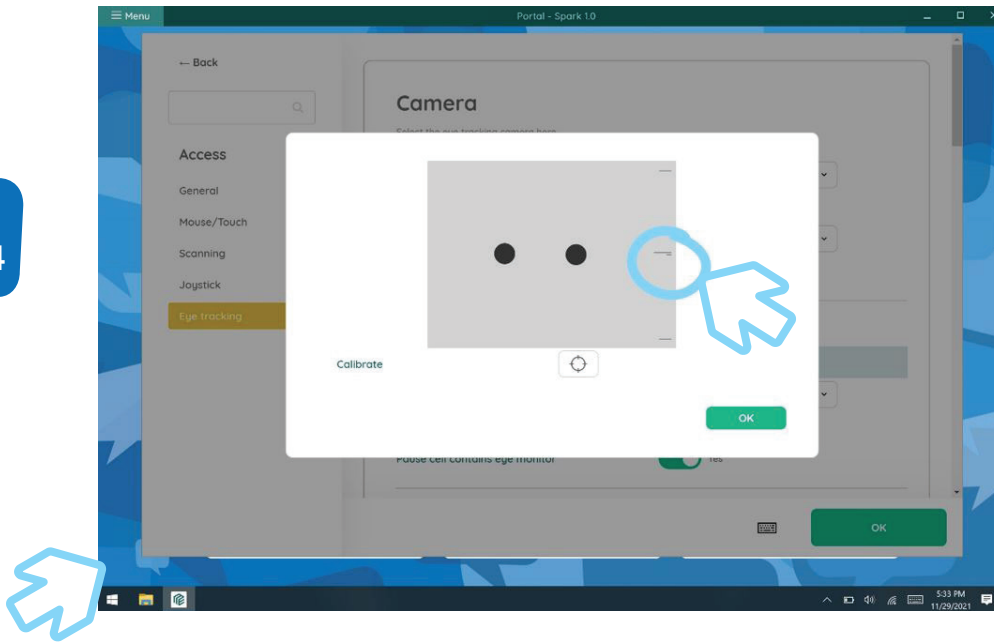

8. Begin calibration by hitting the target button.

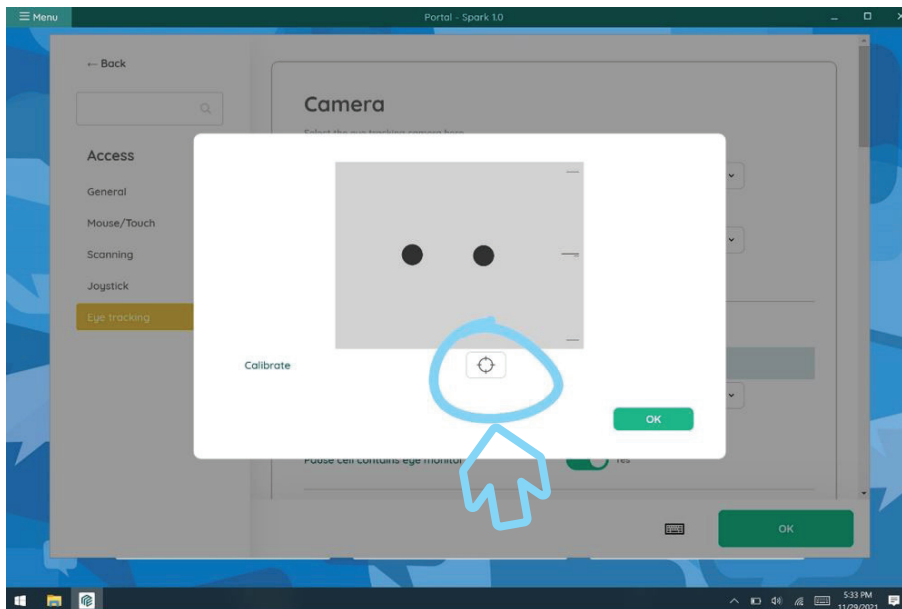


Forbes AAC
181 Illinois Ave. South
Mansfield, OH 44905
phone 419.589.7688
fax 419.589.5146




forbesaac.com

How to
5

*this is not a good calibration. You would want to "Improve"

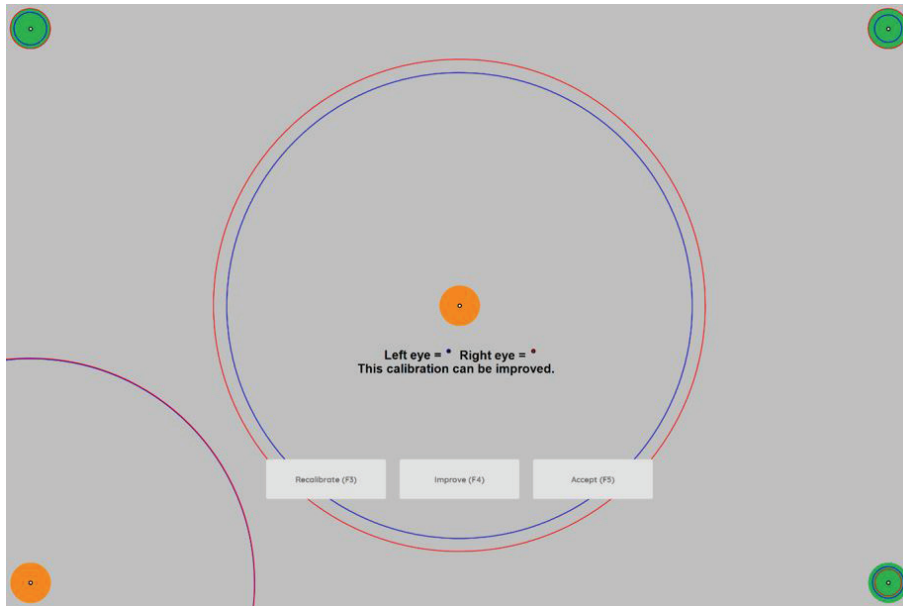

*This is a strong calibration, hit "Accept"

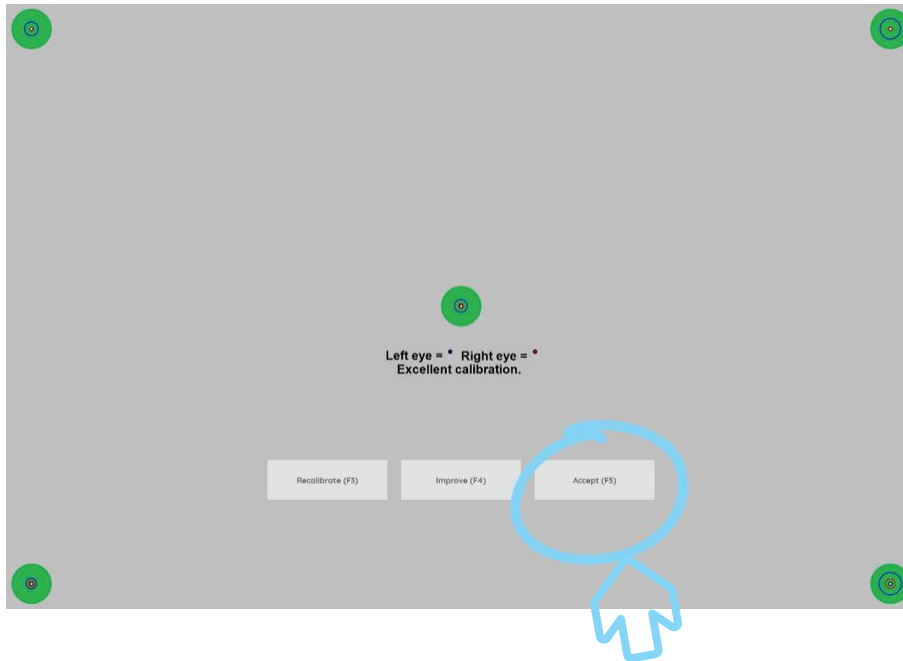


Forbes AAC
181 Illinois Ave. South
Mansfield, OH 44905
phone 419.589.7688
fax 419.589.5146




forbesaac.com

9. Select "OK"

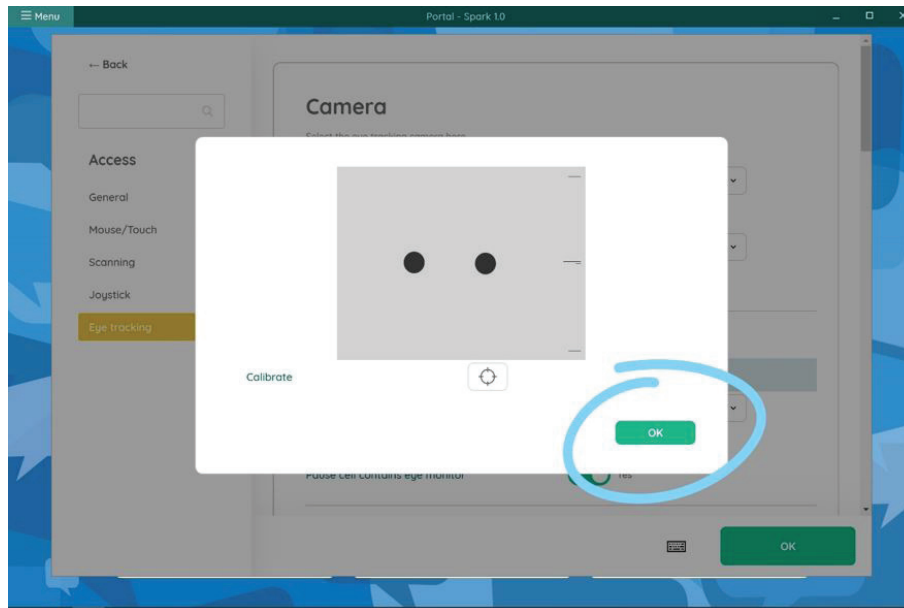

10. Select "OK"

You are ready to begin
EYE TRACKING!

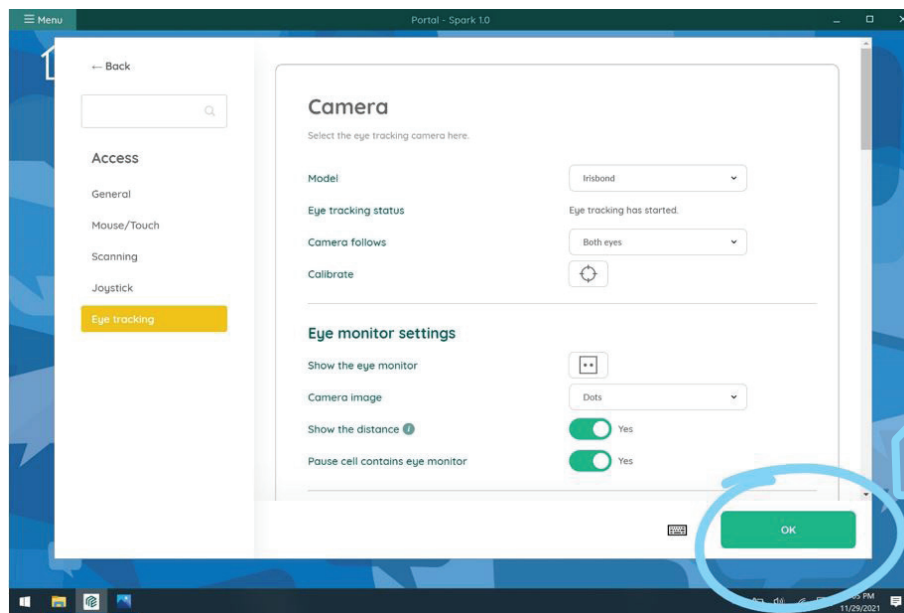

Forbes AAC
181 Illinois Ave. South
Mansfield, OH 44905
phone 419.589.7688
fax 419.589.5146



forbesaac.com