

AAC app cheat sheet

Grid for iPad: Text Talker

Grid Explorer (compass): Dashboard for other grids (Super Core, Amazon Echo, Utilities, etc.)

Back Arrow: Return to previous page

Speak Bubble: Speaks text in message window

... (Three Dots): Opens Edit grids, Find word, and Settings

Chat Icon: Opens Quick Phrases (see image below)

Message Window: Where messages appear

Trash Can: Clears message window

Word Prediction: Reduces keystrokes by making predictions based on single letters typed

Phrase Prediction: Phrase predictions based on phrases previously typed; Use arrows to navigate through predicted phrases

Options

GRIDS	
	Edit grids
	Find word
SETTINGS	
	Settings

123: Numbers and symbols

Gear Icon: Select to see setting options (QWERTY/ABC, text size, brightness, copy, paste, etc.)

M+ and MR: Place to store and retrieve important

TouchChat® is a trademark of PRC-Salttillo

Forbes AAC
 181 Illinois Ave. South
 Mansfield, OH 44905
 419.589.7688
 fax 419.589.5146



forbesaac.com