

AAC app cheat sheet

Predictable

Message Window: Composed message will appear here

Forward Icon: Opens menu of choices to share message (email, copy, past, Twitter, etc.); must set up services prior to use

Speak Icon: Press to say message; Converts to Stop Icon to stop speaking message

Trash Icon: Opens menu to delete letter, word, or all

Voice Emoticons: Opens popup of various voice emoticons

123: Press to open number and symbols; Press ABC to return to keyboard

Gear Icon: Select to open and change settings for voice, accessibility, word prediction, layout, keyboard, and more

Write Icon: Press to open writing area; User can write message with finger or stylus and select corresponding text

Search Icon: Press to open recent messages

Floor Hold: Press to auto play message to communication partner while composing message i.e., "I am almost there", "Please wait a minute"

The interface includes a message window at the top, a keyboard with predictive text (Hello, I, You, We), and various icons for sharing, speaking, deleting, and settings. A grid of voice emoticons is also visible, including Cough, Throat male, Oh, Whistle, Mmm, Throat, Yawn, Laugh, and Kiss male.

Predictable application from AssistiveWare

Forbes AAC
 181 Illinois Ave. South
 Mansfield, OH 44905
 419.589.7688
 fax 419.589.5146



forbesaac.com