


How to set-up your phone with your WinSlate




Enable Bluetooth on **both the phone and the Winslate device** and connect both.

To connect the WinSlate device to Bluetooth

- Go to the menu button (bottom left)

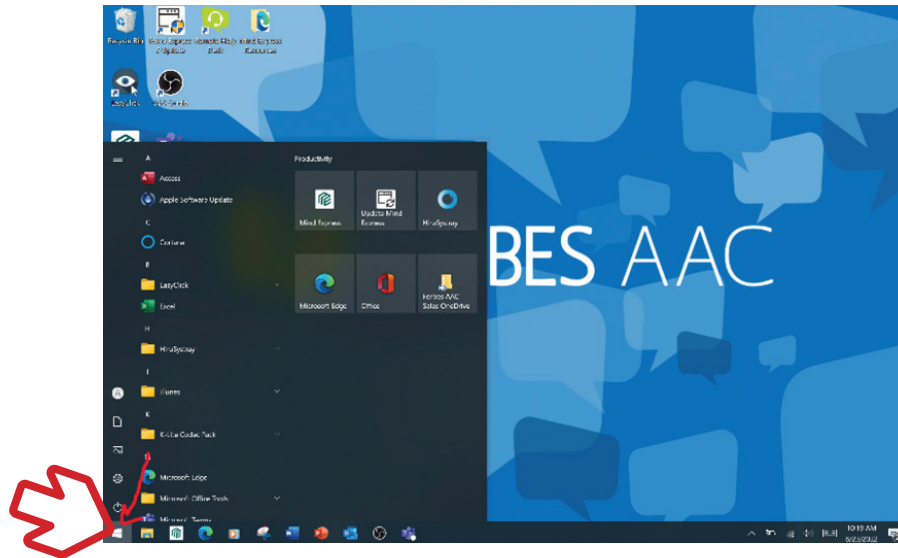

- Go to settings

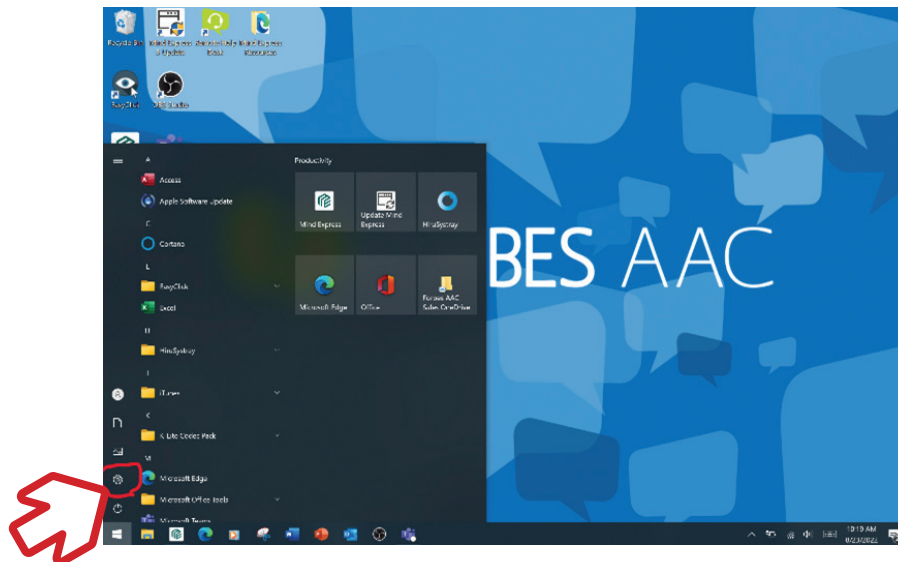


Forbes AAC
181 Illinois Ave. South
Mansfield, OH 44905
phone 419.589.7688
fax 419.589.5146




forbesaac.com

- Go to devices, then make sure “Bluetooth” is toggled on

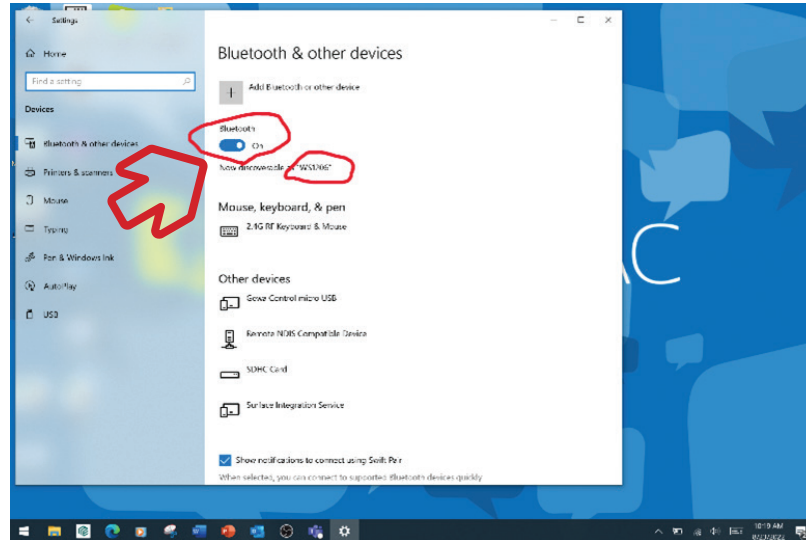

To connect the phone to Bluetooth

- On most phones, go to settings, Bluetooth, and toggle Bluetooth on
- iPhone only: press the 'i' icon next to the paired device in Bluetooth settings on the phone

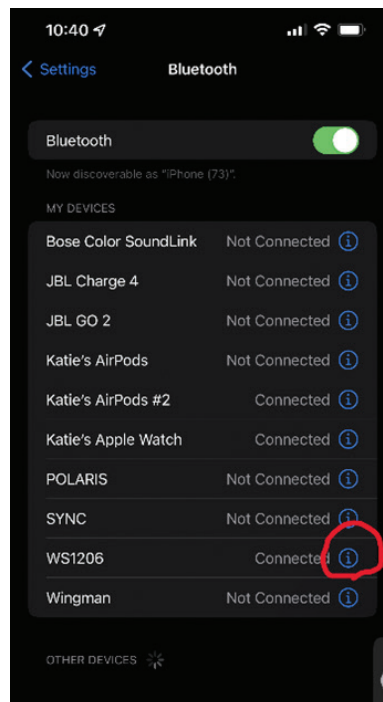

- If that is hidden, go to text communication, mobile phone, then click connect. You should hear a noise to indicate it's connected.




Forbes AAC
 181 Illinois Ave. South
 Mansfield, OH 44905
 phone 419.589.7688
 fax 419.589.5146




forbesaac.com

- Once Bluetooth is on both the WinSlate and the phone, click the plus sign button to "Add Bluetooth or other device"

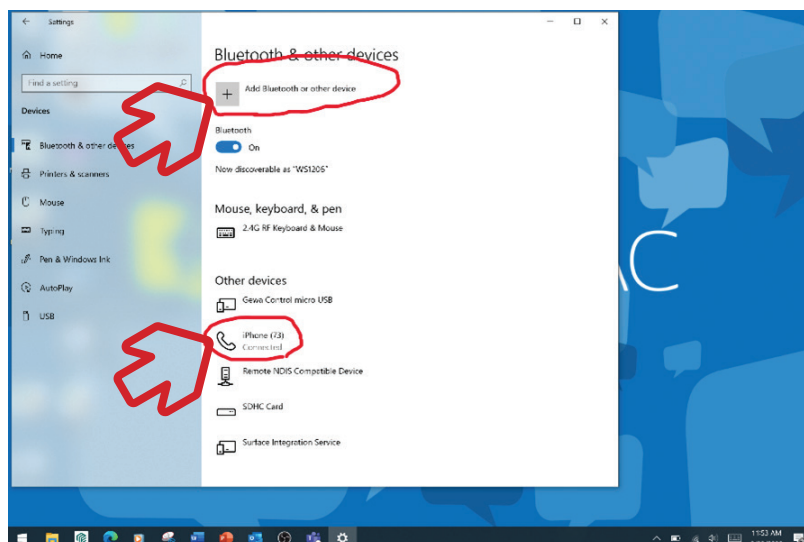

- Open up MindExpress 5
- Go to Menu (top left), then Settings

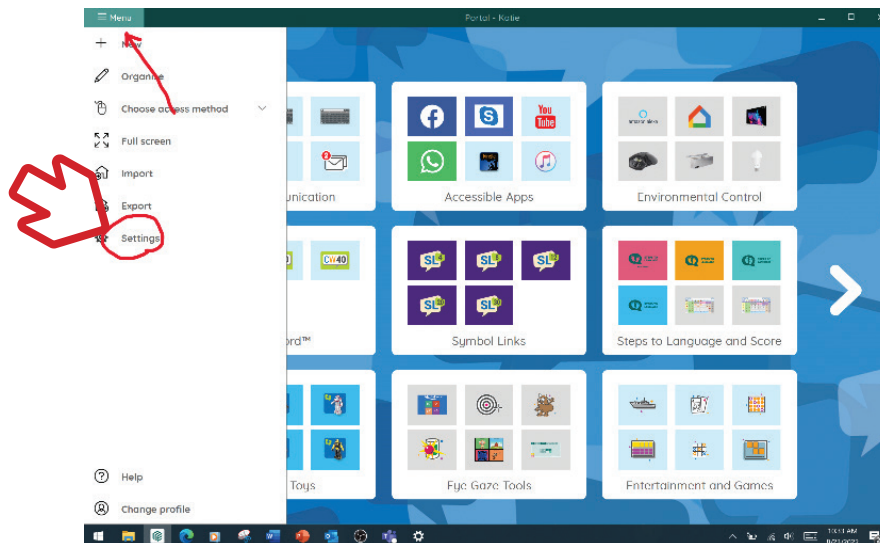


Forbes AAC
 181 Illinois Ave. South
 Mansfield, OH 44905
 phone 419.589.7688
 fax 419.589.5146




forbesaac.com



- Within Settings, choose Devices.

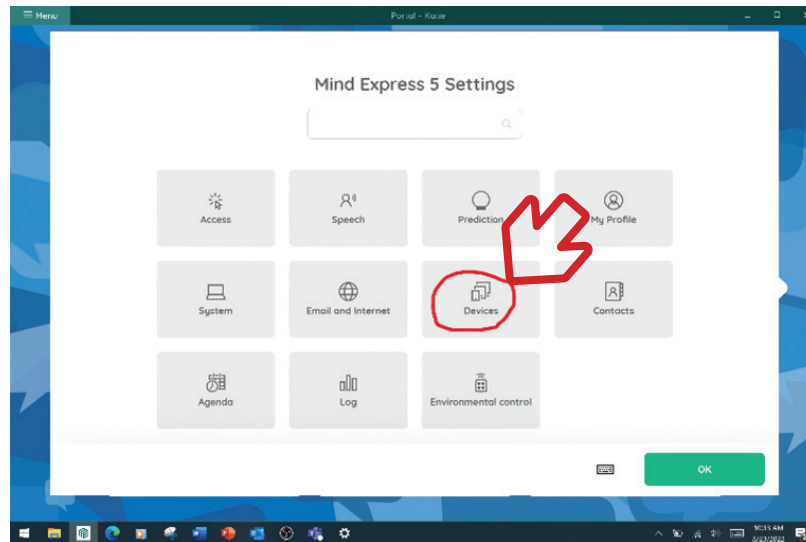

- Choose Phone
- Under Settings, click the "connect with" button
- Choose Bluetooth smartphone from the drop-down list
- Make sure "Connect at startup" is toggled on

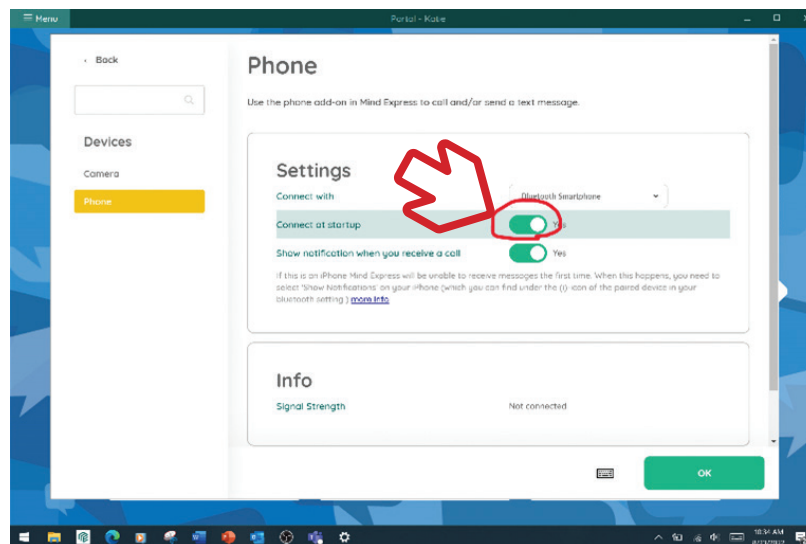

Forbes AAC
181 Illinois Ave. South
Mansfield, OH 44905
phone 419.589.7688
fax 419.589.5146




forbesaac.com

- Once connected, you'll see the phone listed under Info



Sometimes

Spark needs to be closed and re-opened to see that the phone is connected.

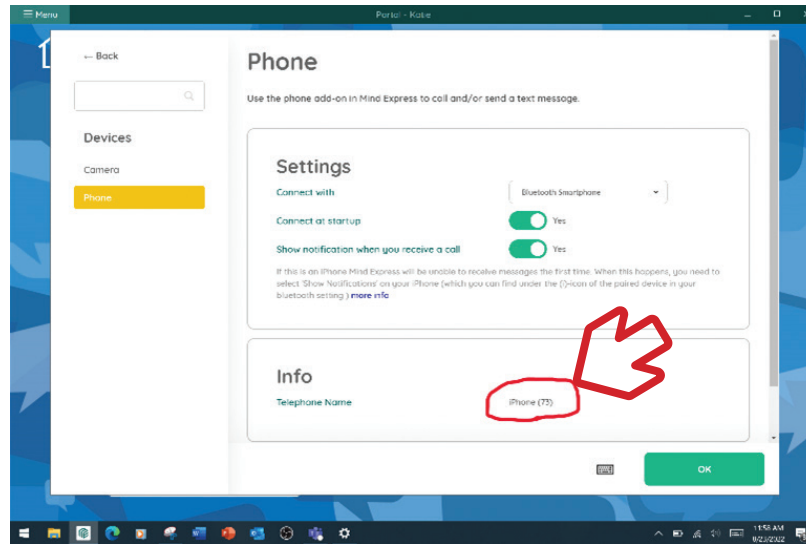

Forbes AAC
181 Illinois Ave. South
Mansfield, OH 44905
phone 419.589.7688
fax 419.589.5146



forbesaac.com